




## Wheelchair Headrest User and Fitting Instructions



The Enigma wheelchair headrest is suitable for retrofitting to most wheelchairs with push handles up to 1" (25mm) diameter.

It features is height, width and angle adjustment and an adjustable padded headrest cushion. It is designed to help aid the posture and comfort of the wheelchair user. It should not be used as part of a restraint in a motor vehicle.

### Installation and Operation

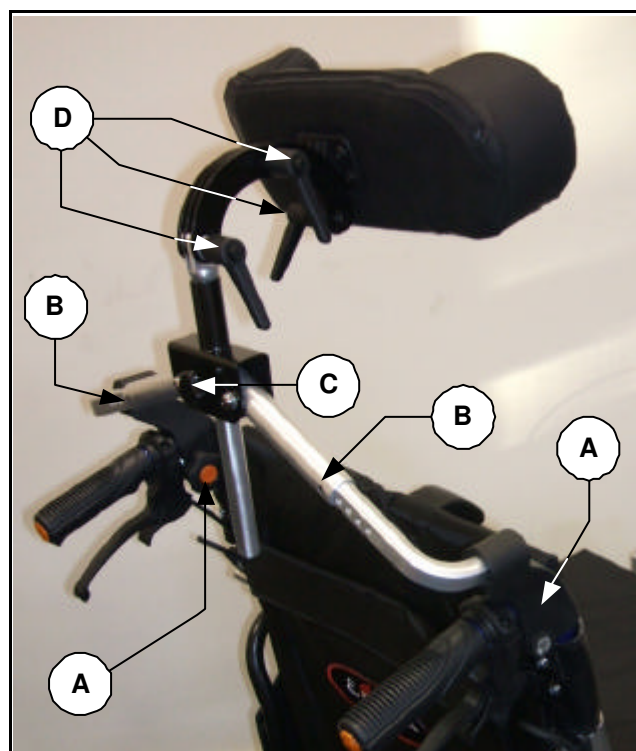

### To Install:

Undo and remove the screws and handwheels attached to the plastic brackets marked 'A'. Loosen the grub screws marked 'B'.

Install the two plastic brackets marked 'A' over the wheelchair push handles, adjusting the width to suit. Secure the brackets by reattaching the handwheels and screws. Secure the headrest width by retightening the grub screws marked 'B'.

### To Adjust the Headrest Height:

Loosen the handwheel marked 'C' by turning anticlockwise. Alter the headrest to the required height and retighten the handwheel.

### To Adjust the Headrest Angle:

Loosen the required handscrews marked 'D' by turning anticlockwise, and adjust the headrest as required. Retighten any loose handscrews.

### Cleaning and Maintenance

- Periodically check the headrest is secured to the wheelchair push handles.
- Check all headrest features function correctly. If problems are found replace the unit.
- The cushion fabric may be cleaned using a damp cloth and mild detergent. Thoroughly dry before use.

### Warranty

The backrest is covered by a twelve-month warranty from date of purchase. It covers design and manufacturing defects. It does not cover upholstery, wear and tear or accidental damage. Please see your dealer for full details of the warranty.

### Specification:

- 36(W) x 7(D) x 12cm(H) – excluding bracketry
- 14¼"(W) x 2¾"(D) x 4¾"(H) – (headrest only)
- Suitable for wheelchair with seat widths 16" (40cm) to 22" (55cm).

### Manufactured by:

Drive Medical Limited  
Ainley's Industrial Estate, Elland,  
West Yorkshire, United Kingdom HX5 9JP.  
Tel: +44 (0) 1422 314488  
Email: enquiries@drivemedical.co.uk  
Web: www.drivemedical.co.uk