VERMEIREN

D200 30°

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

D200 30° - Additions to manual D200

1 Product description (refer to manual D200 paragraph 1)

1.1 Technical specifications (add to manual D200 paragraph 1.2)


Technical terms below are valid for the wheelchair in standard settings. If other footrests / armrests or other accessories are used, the tabulated values will change.

Make	Vermeiren							
Address	Vermeirenplein 1/15, B-2920 Kalmthout							
Type	Manual wheelchair							
Model	D200 30°							
Maximum occupant mass	130 kg							
Description	Dimensions							
Effective seat width	380 mm	400 mm	420 mm	440 mm	460 mm	480 mm	500 mm	520 mm
Overall width (depends on the seat width)	580 mm	600 mm	620 mm	640 mm	660 mm	680 mm	700 mm	720 mm
Description	Dimensions Min.				Dimensions Max.			
Overall length with footrest	1010 mm							
Folded length	1010 mm							
Folded width	300 mm							
Folded height	880 mm				940 mm			
Total mass	16,8 kg							
Mass of heaviest part	9,3 kg							
Static stability downhill	16° (in standard configuration)							
Static stability uphill	9,5° (measured with anti-tipping, place axle block backwards)							
Static stability sideways	20° (in standard configuration)							
Maximum safe slope	12°							
Obstacle climbing	60 mm							
Seat plane angle	5°							
Effective seat depth	430 mm							
Seat surface height at front edge	470 mm				530 mm			
Backrest angle	5°				35°			
Backrest height	420 mm							
Distance between footrest and seat	385 mm				455 mm			
Angle between seat and footplate	-1,5°				18,5°			
Angle between leg footrest and footplate	80°				100°			
Angle between seat and leg footrest	98,5°							
Distance between armpad and seat	220 mm				240 mm			
Front location of armrest structure	410 mm							
Handrim diameter	535 mm							
Horizontal location of axle (deflection)	37 mm							
Minimum turning radius	1500 mm							
Diameter PU Rear wheels	22"				24"			
Diameter PU steering wheels	200 mm							

Storage and use temperature	+ 5 °C	+ 41 °C
Storage and use humidity	30%	70%
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / °		

Table 1: Technical specifications D200 30°

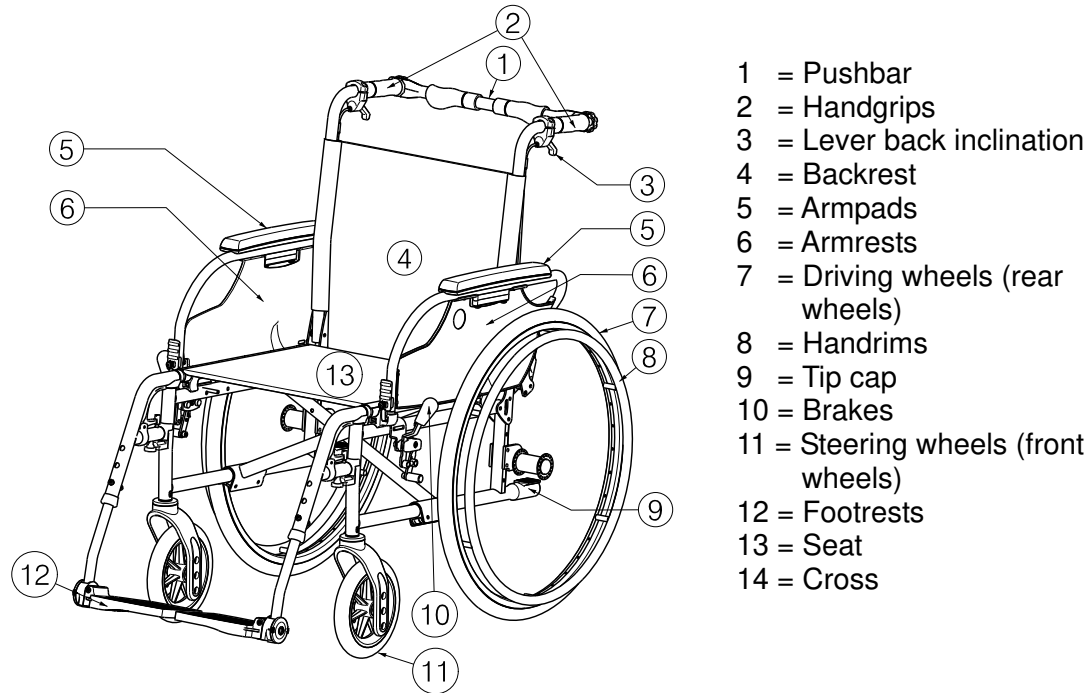1.2 Drawing (change paragraph 1.3 manual D200)

1.3 Accessories (add to manual D200 paragraph 1.4)

The following accessories are available for the D200 30°:

- Drum brakes (B74)

1.4 Explanation of symbols (add to manual D200 paragraph 1.6)

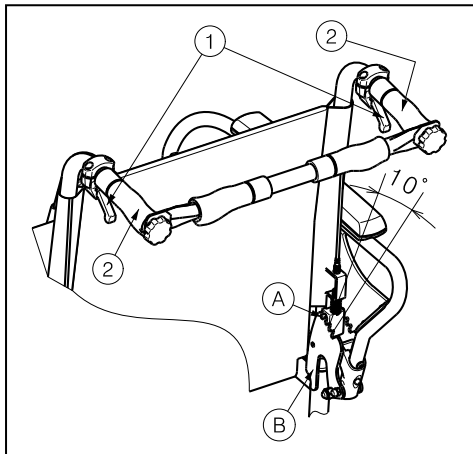
Not crash-tested

2 Use (refer to manual D200 paragraph 2)


2.1 Backrest inclination (add to manual D200)

- ⚠ **WARNING:** Risk of tipping over – Be aware that the stability decreases when the backrest is adjusted backwards.
- ⚠ **WARNING:** Risk of injury – Apply the parking brakes before adjusting the backrest.
- ⚠ **CAUTION:** Chance of pinching – Do not place your fingers, clothes between the adjustment mechanism.

With the model D200 30° it is possible to set the backrest backwards with a maximum inclination of 30° (4 different positions: steps 10°). Be sure that the patient sits in the chair when the attendant shall perform the back adjusting and that the wheelchair does not tip over.

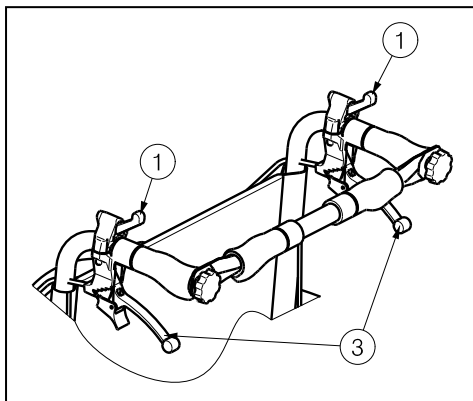
1. Pull lever ① on both sides towards the handgrip ② to adjust the backrest.
2. The pin ① comes out the groove of the toothed plate ②.
3. Pull both back tubes backwards continuously to the desired position (4 different positions: 5° - 15° - 25° - 35°).
4. Check that the pin ① is well fixed in the groove of the toothed plate ②.

If your wheelchair features drum brakes, the levers ③ are used as brakes.

2.2 Mounting or removing of pushbar (add to manual D200)

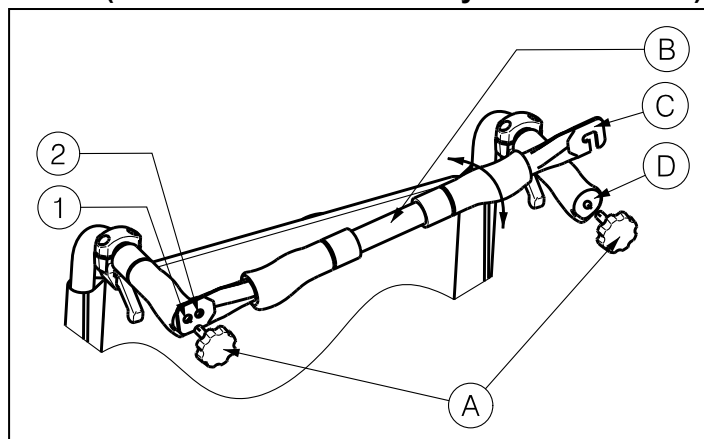- ⚠ **WARNING:** Risk of injury - Make sure that the starknobs ① are tightened.
- ⚠ **WARNING:** Risk of injury - When equipped with the pushbar, the pushbar shall always be mounted (→ to increase the stability of the backrest).

The mounting of the pushbar ① is done as follows:

1. Mount the pushbar ① with the starknob ② in the left handgrip.
2. Swing the pushbar ① to the right.
3. Mount hook ③ over the right handgrip ④ as shown on the drawing. Use holes ① or ② to adjust the tension of the backrest.

To take off the pushbar:

1. Loosen the starknob ② on the right handgrip.
2. Remove hook ③ from the right handgrip ④.

2.3 *Transport in the car* (add to manual D200 paragraph 2.14)

⚠ DANGER: Risk of injury – The wheelchair is not suited for use as a seat in a motor vehicle.

Never use your wheelchair as a seat in an automobile or other vehicle. The wheelchair shall be marked with following symbol.

2.4 *Use of the wheelchair as seat in a motor vehicle* (remove paragraph 2.15)

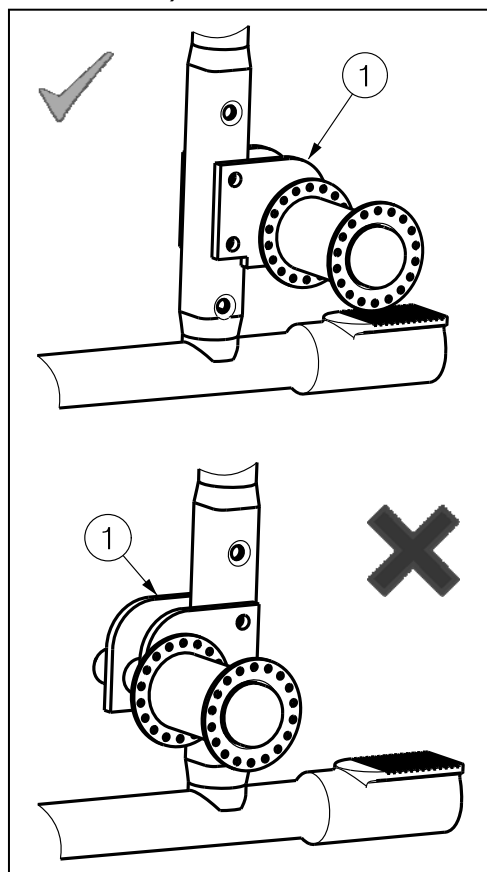
3 Installation and adjustment (refer to manual D200 paragraph 3)

3.1 *Manner of delivery* (add to manual D200 paragraph 3.2)

The Vermeiren D200 30° shall be delivered with:

- 1 frame with armrests, rear and front wheels (standard delivery seat height: 500 mm, seat angle: 5°)
- 1 pair footrests
- Back adjusting mechanism
- Tools
- Manual
- Accessories

3.2 *Adjusting the stability and the maneuverability* (add to manual D200)

Place axle block ① in most backwards position.

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Vermeiren Suisse S.A.