

rossmax

just a heartbeat away

rossmax



血氧濃度計傳輸軟體安裝教學 Rossmax Pulse oximeter installation SOP

首次安裝需安裝「安全憑證」

Need to install "SSL Certificate" for first time


| Name | Date modified | Type | Size |
|--|--------------------|----------------------|----------|
| Add-AppDevPackage.resources | 2/18/2019 1:07 AM | File folder | |
| Dependencies | 2/18/2019 1:07 AM | File folder | |
| Add-AppDevPackage | 12/20/2018 9:49 PM | Windows PowerS... | 34 KB |
| Pulse_Oximeter_Bluetooth_Connect_2_1.... | 2/18/2019 1:07 AM | APPXSYM File | 1,973 KB |
| Pulse_Oximeter_Bluetooth_Connect_2_1.... | 2/18/2019 1:07 AM | APPXBUNDLE File | 2,492 KB |
| Pulse_Oximeter_Bluetooth_Connect_2_1.... | 2/18/2019 1:07 AM | Security Certificate | 1 KB |



點擊開啟安裝憑證

Please click this to install SSL certificate

點擊安裝憑證 Click install certificate

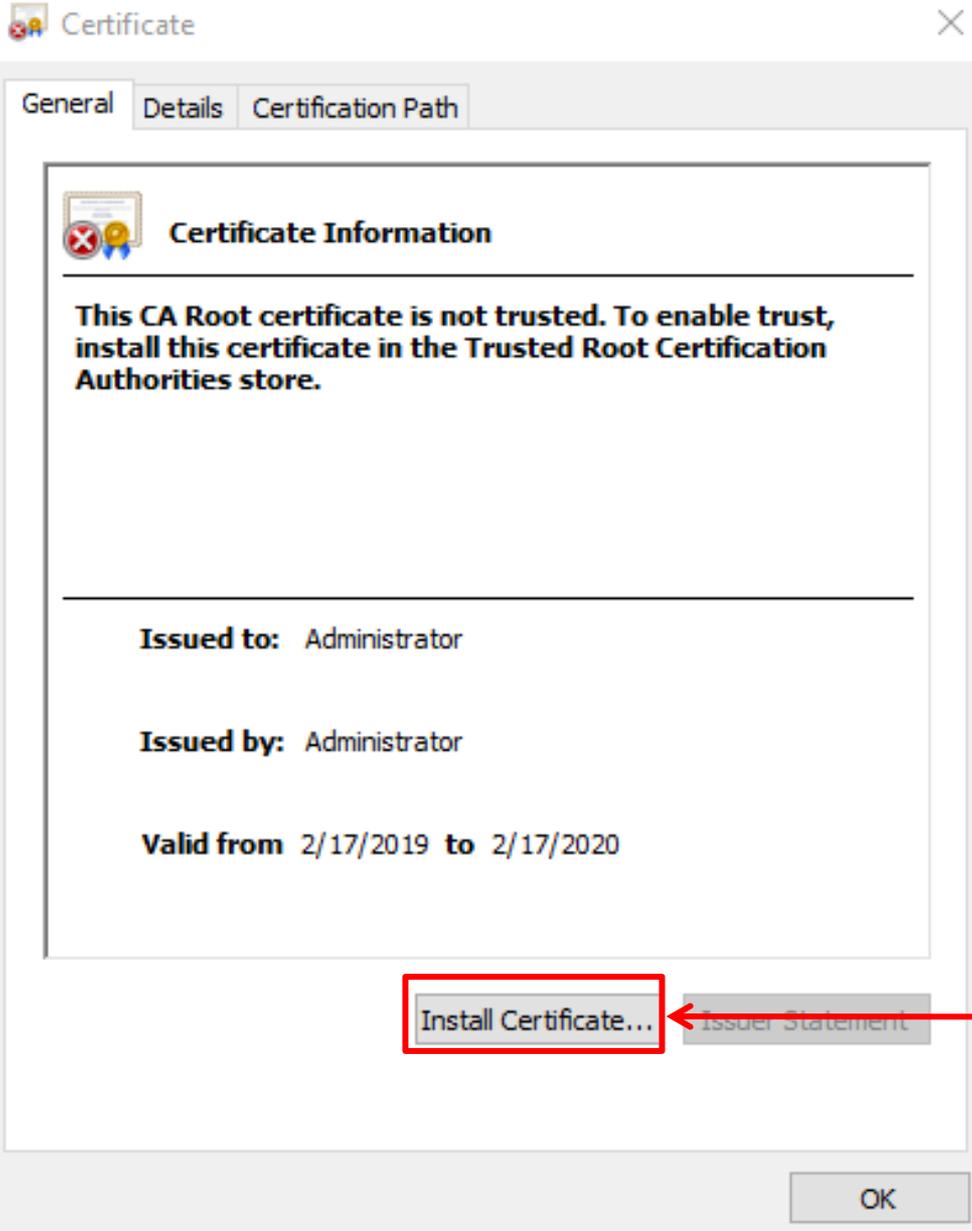

點擊開啟安裝憑證
Click install “SSL”
bottom

← Certificate Import Wizard

Welcome to the Certificate Import Wizard

This wizard helps you copy certificates, certificate trust lists, and certificate revocation lists from your disk to a certificate store.

A certificate, which is issued by a certification authority, is a confirmation of your identity and contains information used to protect data or to establish secure network connections. A certificate store is the system area where certificates are kept.

Store Location

Current User

Local Machine

← 1. 點選本機電腦


Choose this "Local Machine"

To continue, click Next.


2. 點選下一步
Click next step

安全憑證-憑證存放位置

SSL-the storage of all certificate

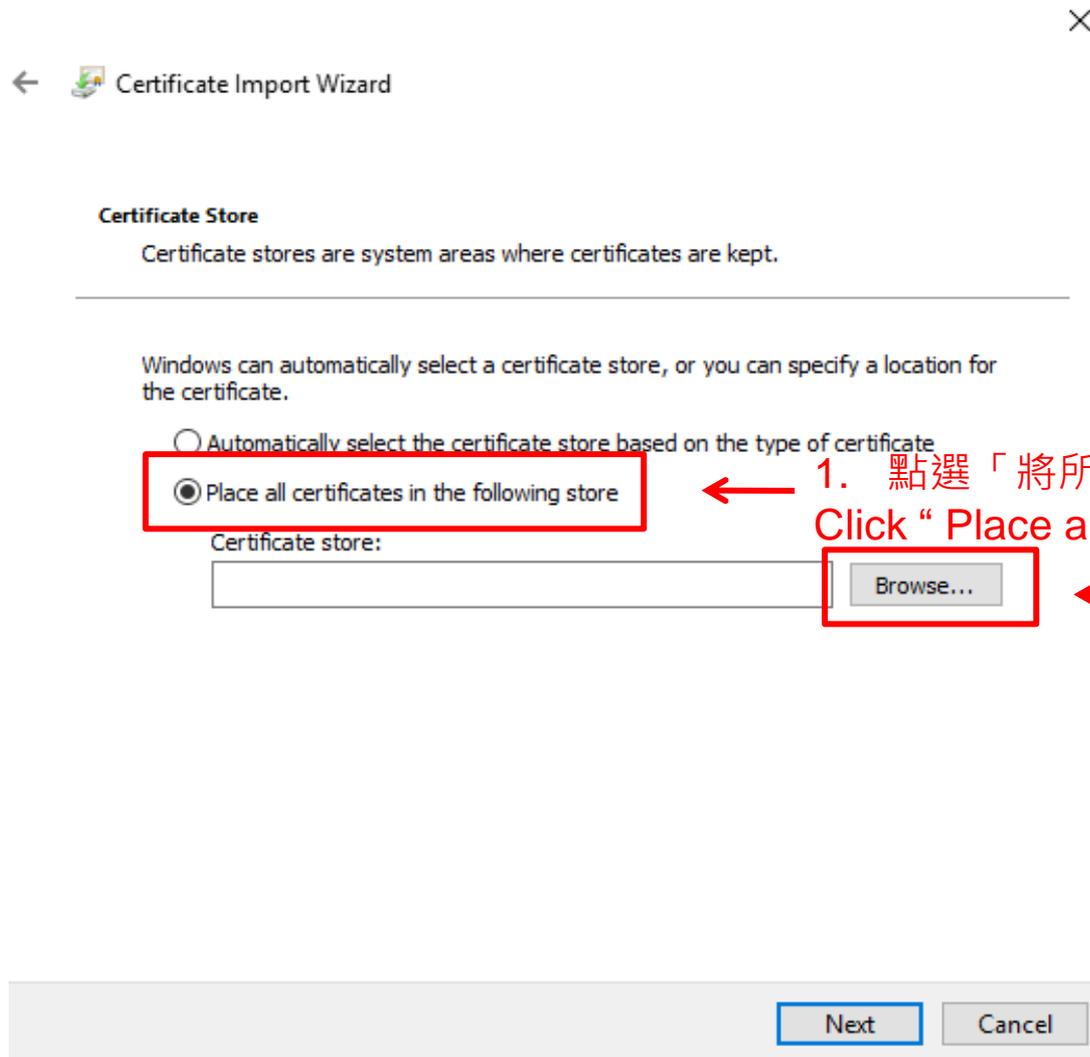

1. 點選「將所有憑證放入以下的存放區(P)」
Click "Place all certificates in the following store"
2. 點選「瀏覽」
Click "Browse"

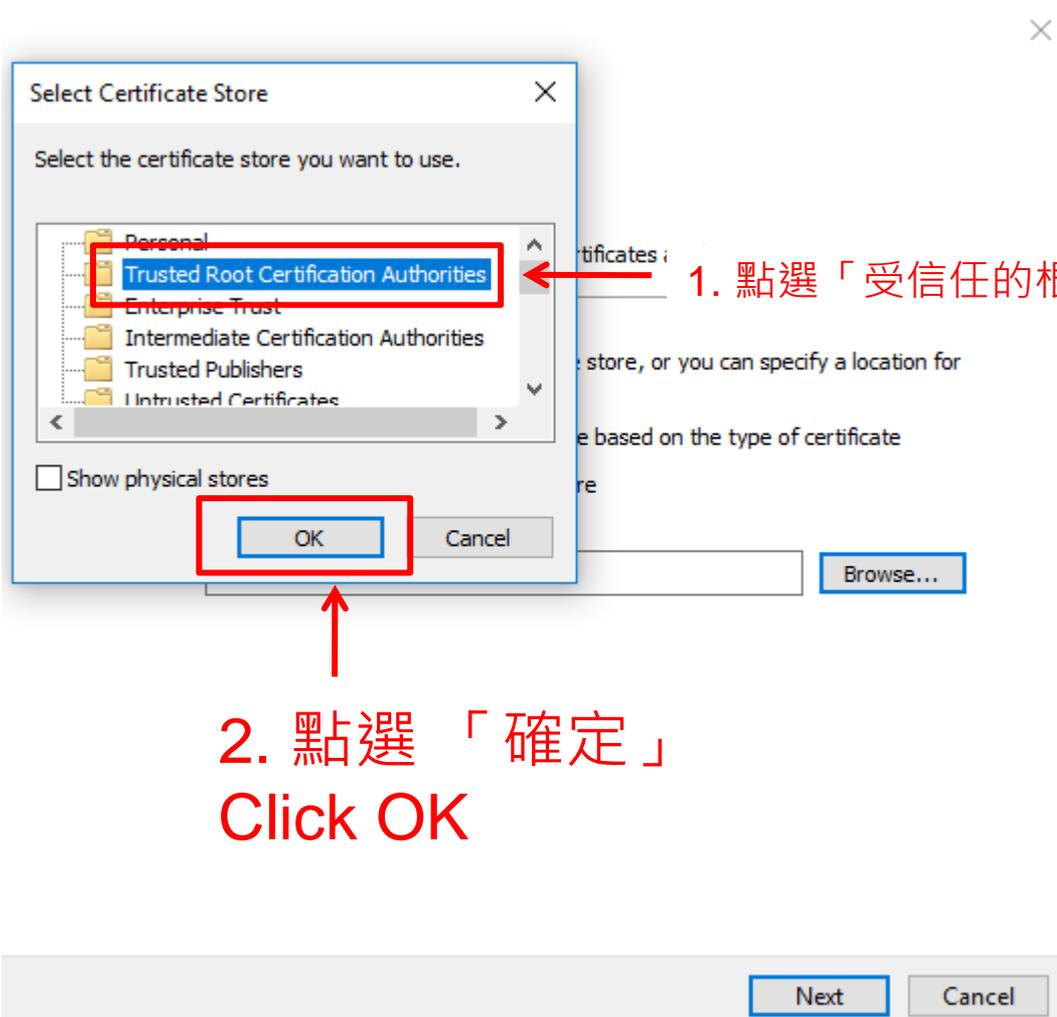


1. 點選「受信任的根憑證授權單位」 Click this one.

2. 點選「確定」
Click OK

安全憑證 – 確認存放區

SSL-Make sure the storage

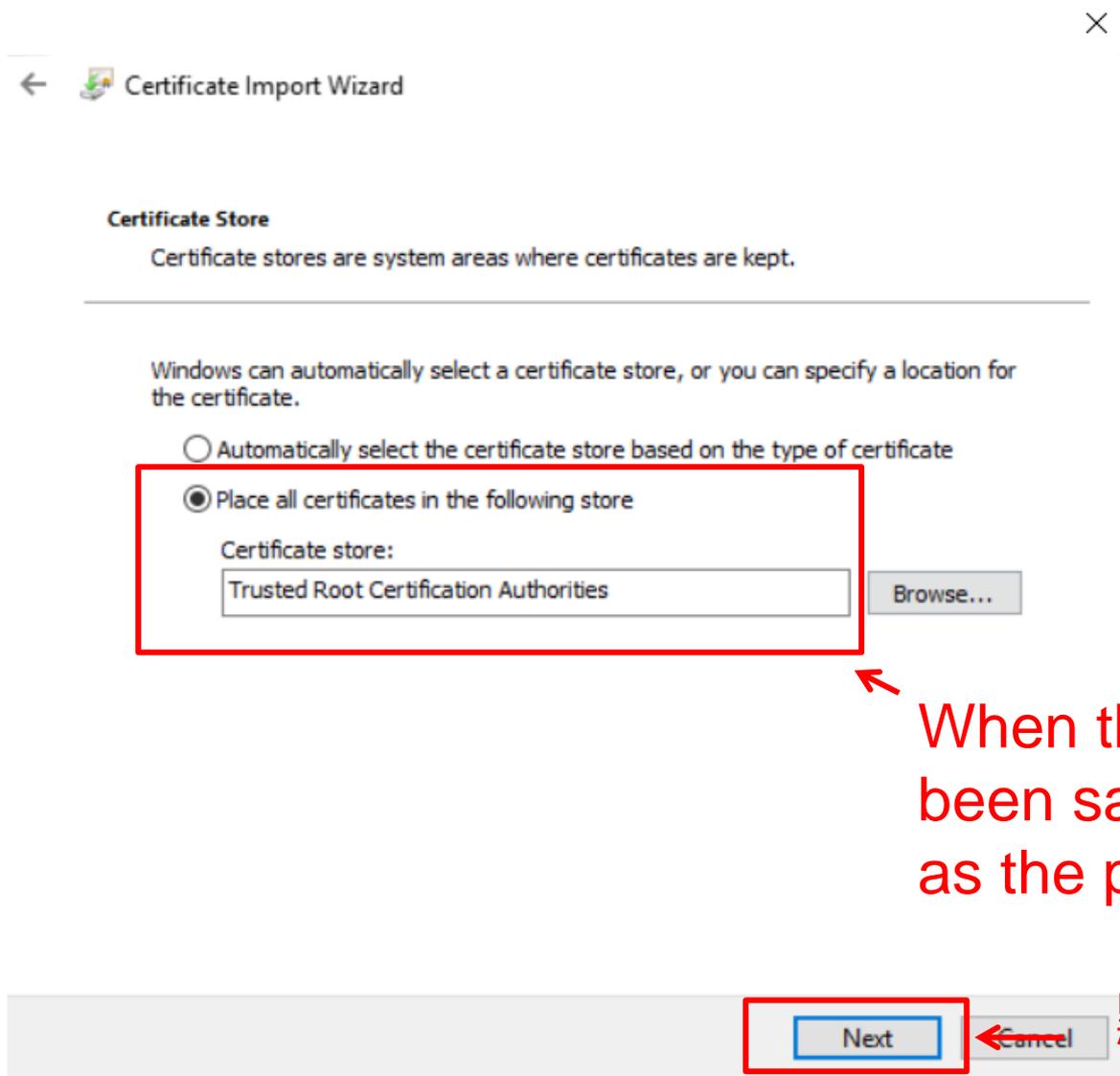

When the certificate has been save to correct place, as the picture.

點選「下一步」
Click Next

← Certificate Import Wizard

Completing the Certificate Import Wizard

The certificate will be imported after you click Finish.

You have specified the following settings:


| | |
|------------------------------------|--|
| Certificate Store Selected by User | Trusted Root Certification Authorities |
| Content | Certificate |



點擊「完成」
Click Finish

安全憑證- 匯入執行成功

SSL-Import success

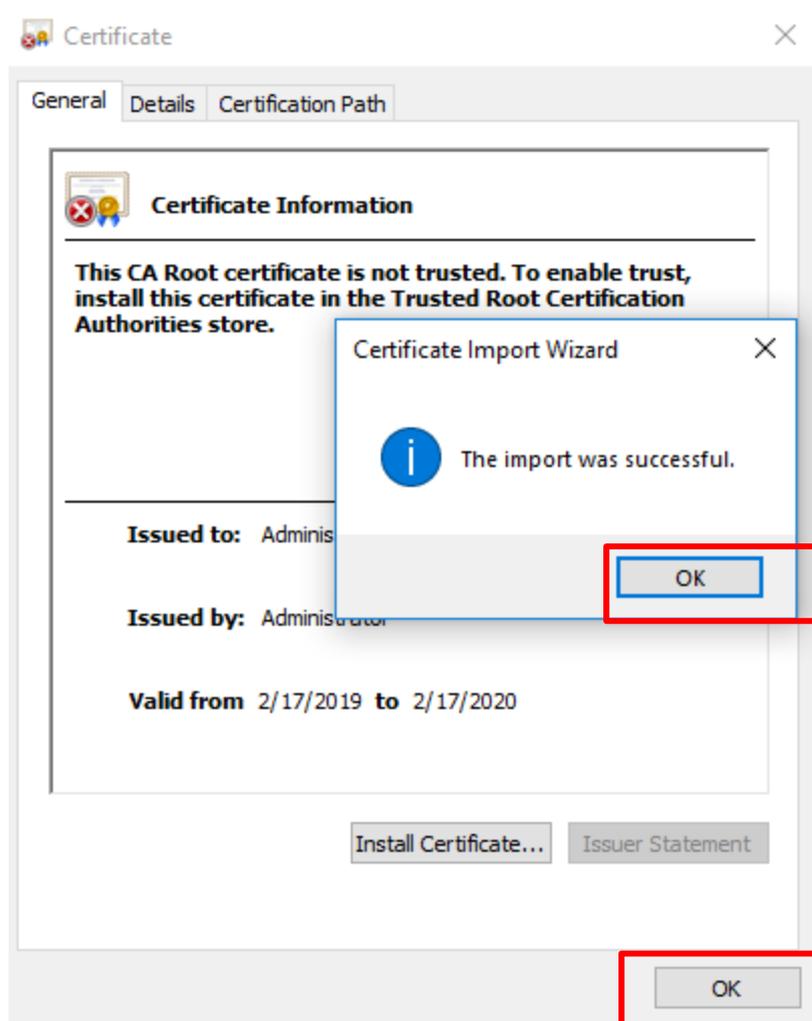

1. 點擊「確定」
Click OK

2. 點擊「確定」
Click OK

點選安裝執行檔 Install Software

| Name | Date modified | Type | Size |
|--|--------------------|----------------------|----------|
| Add-AppDevPackage.resources | 2/18/2019 1:07 AM | File folder | |
| Dependencies | 2/18/2019 1:07 AM | File folder | |
| Add-AppDevPackage | 12/20/2018 9:49 PM | Windows PowerS... | 34 KB |
| Pulse_Oximeter_Bluetooth_Connect_2_1.... | 2/18/2019 1:07 AM | APPXSYM File | 1,973 KB |
| Pulse_Oximeter_Bluetooth_Connect_2_1.... | 2/18/2019 1:07 AM | APPXBUNDLE File | 2,492 KB |
| Pulse_Oximeter_Bluetooth_Connect_2_1.... | 2/18/2019 1:07 AM | Security Certificate | 1 KB |

點擊開啟安裝安裝執行檔

Click software as APPXBUNDLE to install.

安裝執行檔畫面

Install Software display

Install Pulse_Oximeter_Bluetooth_Connect_2?

Publisher: Administrator

Version: 1.0.2.0

Capabilities:

- Close themselves and their own windows, and delay the closing of their app

Launch when ready



← 點擊「安裝」
Click "install"

傳輸軟體安裝成功

Installation finished as below

